






## Regeneten preparation

Things need to open:

Product Ref:	Product name:
4402	TENDON STABILIZING GUIDE (1) 
4565	BIOINDUCTIVE IMPLANT W/ ARTHRO DEL PKG MED 
4403	BONE ANCHOR (3) W/ ARTHRO DEL SYS ADVANCED 
2504-1	TENDON ANCHOR (8) 

**Things need to be prepared by nurses after opening those 4 boxes:**

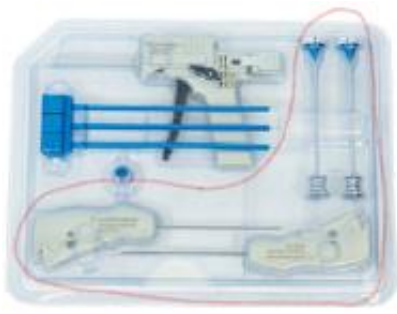


BONE ANCHOR (3) W/ ARTHRO DEL SYS ADVANCED (Ref: 4403)



TENDON ANCHOR (Ref: 2504-1)

- 1) Take these out (Tendon anchor inserter x2, cannula x2, trocar x2 and Tendon anchor box)

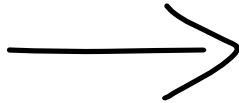

- 2) Insert the trocars into cannulas



- 3) **Fully insert** tendon anchor inserter into the hole of box

**Press the button** on the inserter to engage the purple tendon anchor with the inserter

Repeat this step for another inserter and everytime after Doctor use on patient

**Surgery steps preparation:**



- 1) Tendon stabilizing guide (Ref: 4402) + Mallet



- 2) BIOINDUCTIVE IMPLANT W/ ARTHRO DEL PKG MED (Ref: 4565)




- 3) Combined cannula (From Ref: 4403) +  
Loaded tendon anchor inserter (From Ref: 4403 with tendon anchor from Ref: 2504-1)




- 4) Bone anchor delivery gun and Bone anchor (From Ref: 4403) + Mallet


**If need to insert more than 1 bone anchor, please reset the delivery gun:**

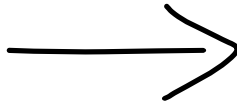

1) After Doctor inserted the bone anchor, **remove** the Blue inserter

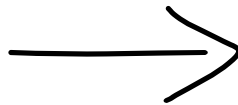

2) Insert back the White inserter to the gun, press the fire button to half position

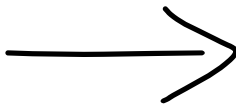

3) Push the top forward to reset the outer sheath of the gun